

NOTE: For access to the full user manual for this product, please visit www.heska.com/productmanuals.

Vet/IV™ 2.2 Infusion Pump Overview

Fluid Choices:

- Electrolyte solutions
- Glucose-containing solutions
- Total Parenteral Nutrition (TPN)

Do not use to administer plasma or whole blood

IV Set Choices:

Hospira calibration:

- Abbott Series A
- Hospira IV Sets
- Zoetis Series A
- Henry Schein UniverSet


Baxter calibration:

- Abbott Series B
- Baxter Brand IV Sets
- Zoetis Series B

Use calibrated brand only.

Do not interchange.

Use standard sets for MACRO mode; micro sets for MICRO mode.

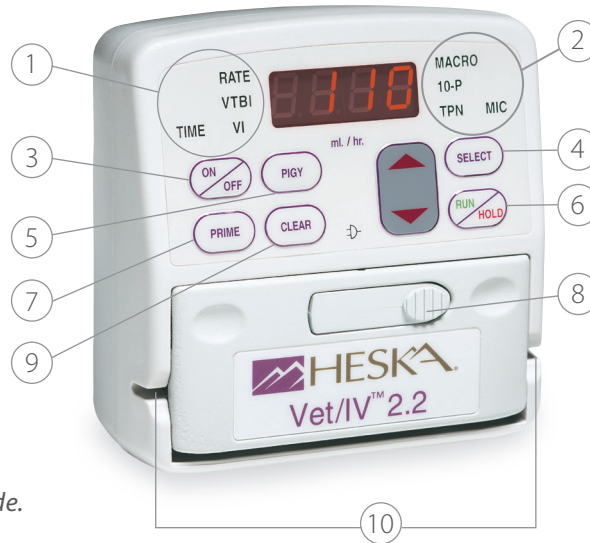

Front Panel

1. User-definable parameters:
 - Rate of infusion (RATE)
 - Volume to be infused (VTBI)
 - Time of infusion
2. Infusion mode options:
 - MACRO
 - 10-P
 - MICRO
 - TPN
3. ON/OFF button
4. Select infusion mode, pressure
5. Piggyback infusion mode
6. RUN program or put on HOLD
7. Prime lines
8. Door latch
9. Clear programming
10. Tubing guide

For questions or assistance in an emergency, call Heska's Technical Support Services at 800.464.3752, option 3.

MACRO and MICRO Infusion Setup

1. Plug pump into AC wall outlet or use on battery power.
2. Open front door and insert IV line into guide groove so that fluids flow from left to right. Close door.
3. Hold **SELECT** down, then quickly press and release **ON/OFF**. Release **SELECT**. [SLCT] appears on display.
4. Press **SELECT** until MACRO or MICRO indicator turns green, then press **RUN/HOLD**.
5. If needed, press **SELECT** to scroll to desired pressure setting (HI, NOR, LO) then press **RUN/HOLD**.
6. RATE indicator turns green. Press ▲ or ▼ to scroll to desired rate of administration, then press **RUN/HOLD**.
7. You may now set either the total time of administration or the volume to be infused. Press **SELECT** to highlight either TIME or VTBI, then use ▲ or ▼ to scroll to desired setting. Press **RUN/HOLD**.
8. Press **PRIME** to fill IV lines with fluid.
9. Press **RUN/HOLD** to begin delivery.
10. To stop delivery, press **RUN/HOLD**.

Checking Settings

- To check time remaining, press **SELECT** once.
- To check remaining volume to be infused, press **SELECT** twice.
- To check volume that has been infused, press **SELECT** 3 times.

Adjusting Settings

- Press **RUN/HOLD**.
- Press **SELECT** until parameter to be adjusted is displayed.
- Use ▲ or ▼ to adjust setting.
- Press **RUN/HOLD** to resume operation.

Other Infusion Options

10-P: Sets up to 10 different rate/volume combinations to be given to a single patient over time.

PIGY: Allows two separate rate/volume settings to be programmed to accommodate a main fluid bag plus a piggybacked supplemental bag.

TPN: Programs ramp-up, ramp-down and total delivery time for use with Total Parenteral Nutrition.

See Heska Vet/IV™ 2.2 Infusion Pump User Manual for details.