


How to Find Studies in HeskaView+

HeskaView+ has many combinations of search criteria to use to find a study.

Upon logging in, the screen to the right will appear.



1. Enter any search criteria in the default search condition, or change the drop down menus to other criteria.
 - a. Enter the preferred search criteria into the corresponding box beneath the drop down menu, and click  to search.
 - b. Select  to clear the search fields.
 - c. Drag and drop the search result columns as desired or right click on them to add or remove fields.

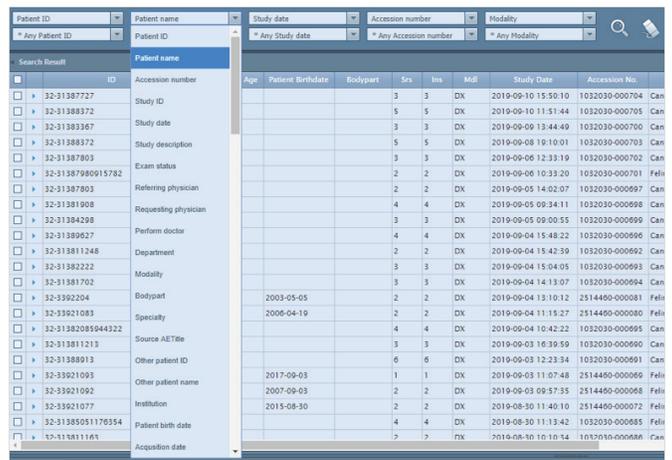

- d. Click **SAVE SETTINGS**, top right, for the viewer to remember preferences.

2. To open a study, double click the study or select .

