

How to Add a New Browser Bookmark in HeskaView+

Navigate to <https://www.heska.com/HeskaViewPlus>

Add the new desktop shortcuts, and/or browser favorites for HeskaView+.

NOTE: Do not delete your old bookmarks for ViewCloud. They will be used to temporarily access your previous images while migrating your data to the new platform.

Modalities Update

Don't worry, the Cuattro DR digital imaging settings will update automatically. If you have other imaging devices that use ViewCloud, which are not Heska products, please contact your vendor and have them update the following settings:

Example of Old Settings	New Settings
AE Title:	
MPACS01 or MDSRV1	NETNOTLS
IP Address:	
PRO01.CUATTRO.COM or MDSRV1. CUATTRO.COM	heskanet2.infinittna.com
Port:	
104	104
Client (Sending) AE Title:	
DXXSXXXXXXXX	Do not change. This should stay in the current DXXSXXXXXXXX format.

NOTE: For Esaote products please call: 800.428.7378.