

NEW



trūRapid **HW** (Dirofilaria)

Introducing Heska® trūRapid™ Heartworm - part of the latest point-of-care infectious disease testing developed for all practices.

Focus on canine wellness with fast, accurate, and easy-to-use heartworm testing. The trūRapid Heartworm (Dirofilaria) antigen test has everything a veterinary professional could want in a point-of-care heartworm test.



10 minutes to results

Room temperature storage

Excellent specificity and sensitivity

Long shelf life

www.heska.com




trūRapid HW

Rapid In-house Heartworm Detection

...with accurate results in 10 minutes.

**Prevention, prevention, prevention.
Don't let your patients reach this stage!**


Heartworm incidence is increasing across the country, so comprehensive prevention programs are a must. Utilize trūRapid Heartworm Tests as part of your prevention program, and provide gold standard diagnostics to keep your patients healthy and clients happy.

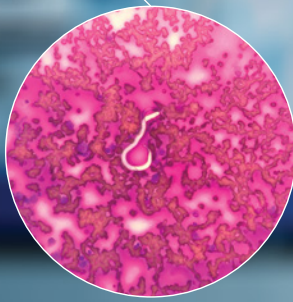

trūRapid™ Heartworm Study			
	Sample True Status		
	Positive	Negative	
trūRapid	Positive	147	0
Heartworm Test	Negative	3	200
Sensitivity	98.0% (95% CI: 94.3–99.6%)		
Specificity	100.0% (95% CI: 98.2–100.0%)		

**Accuracy, Reliability, Speed,
and Ease-of-Use ...**


- Long shelf life with no refrigeration needed
- 10 minutes to results
- Excellent specificity/sensitivity
- Single, easy-to-use cassettes with no need to snap
- Utilize anti-coagulated whole blood, serum, or plasma

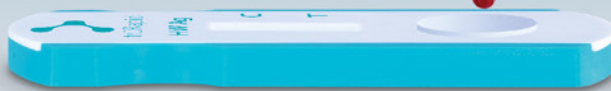


Upcoming trūRapid tests ...

- **trūRapid FIV/FeLV**
Feline Immunodeficiency Virus/ Feline Leukemia Virus
- **trūRapid FOUR**
Heartworm (Dirofilaria), Ehrlichia spp, Anaplasma spp, Borrelia burgdorferi
- **trūRapid Giardia**
Giardia duodenalis
- **trūRapid Parvovirus**
Canine Parvovirus

SIMPLE 3 STEP PROCESS

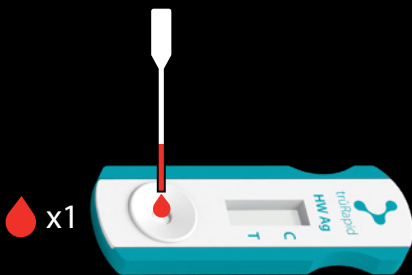

1 Dispense 1 drop of sample into test well

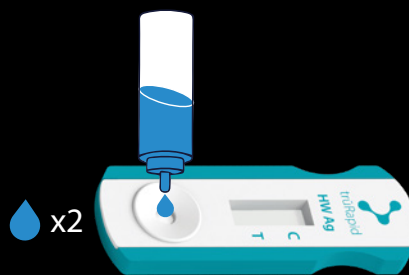

2 Dispense 2 drops of Chase Buffer into test well

10 mins

Valid



Invalid



3 Interpret the results ...