

NEW



## trūRapid **HW** (Dirofilaria)

Introducing Heska® trūRapid™ Heartworm - part of the latest point-of-care infectious disease testing developed for all practices.

Focus on canine wellness with fast, accurate, and easy-to-use heartworm testing. The trūRapid Heartworm (Dirofilaria) antigen test has everything a veterinary professional could want in a point-of-care heartworm test.



10 minutes to results

Room temperature storage

Excellent specificity and sensitivity

Long shelf life

[www.heska.com](http://www.heska.com)



# trūRapid HW

## Rapid In-house Heartworm Detection

...with accurate results in 10 minutes.

**Prevention, prevention, prevention.  
Don't let your patients reach this stage!**

Heartworm incidence is increasing across the country, so comprehensive prevention programs are a must. Utilize trūRapid Heartworm Tests as part of your prevention program, and provide gold standard diagnostics to keep your patients healthy and clients happy.



trūRapid™ Heartworm Study			
	Sample True Status		
	Positive	Negative	
trūRapid	Positive	147	0
Heartworm Test	Negative	7	200
Sensitivity	98.0% (95% CI: 94.3–99.6%)		
Specificity	100.0% (95% CI: 98.2–100.0%)		

### Accuracy, Reliability, Speed, and Ease-of-Use ...


- Up to one year shelf life with no refrigeration needed
- 10 minutes to results
- Excellent specificity/sensitivity
- Single, easy-to-use cassettes with no need to snap
- Anti-coagulated whole blood, serum, or plasma




### Upcoming trūRapid tests ...

- **trūRapid FIV/FeLV**  
Feline Immunodeficiency Virus/ Feline Leukemia Virus
- **trūRapid FOUR**  
Heartworm (Dirofilaria), Ehrlichia spp, Anaplasma spp, Borrelia burgdorferi
- **trūRapid Giardia**  
Giardia duodenalis
- **trūRapid Parvovirus**  
Canine Parvovirus

## SIMPLE 3 STEP PROCESS

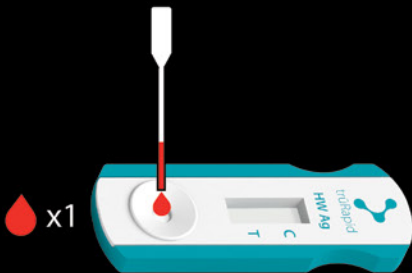

**1** Dispense 1 drop of sample into test well

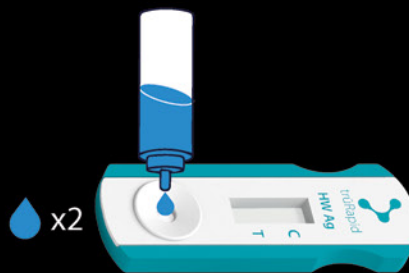

**2** Dispense 2 drops of Chase Buffer into test well

10 mins

Valid



Invalid



**3** Interpret the results ...