


NEW



## trūRapid FIV/FeLV

Introducing the Heska® trūRapid™ FIV/FeLV combination test.

Developed for all practices, it joins the family as our latest in-house infectious disease test. Focus on feline wellness with fast, accurate, and easy-to-use feline viral testing. The trūRapid FIV/FeLV test has everything a veterinary professional could want, in a point-of-care viral test.

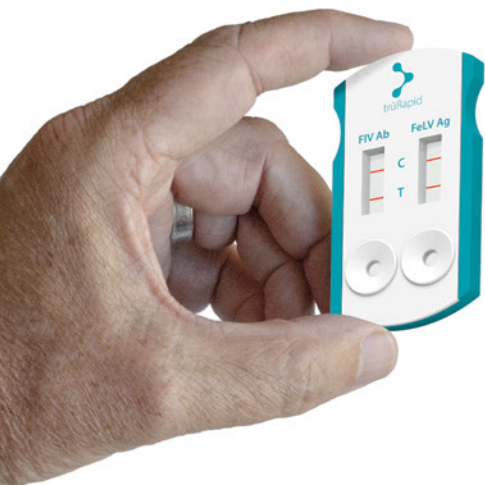

10 minutes to results

Room temperature storage

Excellent specificity and sensitivity

Long shelf life



[www.heska.com](http://www.heska.com)




# trūRapid FIV/FeLV

Rapid In-house FIV/FeLV Detection

...with accurate results in 10 minutes.

**Help stop the spread of these cancer-causing viruses with regular screening and early disease detection.**

Feline leukemia virus and feline immunodeficiency virus are common causes for cancer and a variety of other illnesses in feline patients. Early identification remains an important factor for treating secondary illnesses and preventing new infections. Employ gold standard diagnostics by utilizing the trūRapid FIV/FeLV test as a part of your screening and prevention programs.

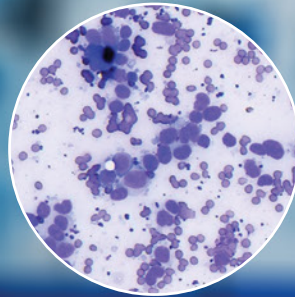

Feline Intestinal Lymphoma  
telecytology image from:  
**heskaView**



trūRapid™ FIV Study			
		IFA	
		Positive	Negative
trūRapid	Positive	102	2
FIV Antibody	Negative	1	134
Sensitivity		99.0% (95% CI: 94.7–100%)	
Specificity		98.5% (95% CI: 94.8–99.8%)	

trūRapid™ FeLV Study			
		ELISA	
		Positive	Negative
trūRapid	Positive	99	0
FeLV Antigen	Negative	1	135
Sensitivity		99.0% (95% CI: 94.6–100%)	
Specificity		100.0% (95% CI: 97.3–100.0%)	

## Accuracy, Reliability, Speed, and Ease-of-Use ...

- Long shelf life with no refrigeration needed
- 10 minutes to results
- Excellent specificity/sensitivity
- Single, easy-to-use cassettes with no need to snap
- Utilize anti-coagulated whole blood, serum, or plasma

## Upcoming trūRapid tests ...

- **trūRapid HW/Lyme**  
Heartworm (Dirofilaria)/ Borrelia burgdorferi
- **trūRapid FOUR**  
Heartworm (Dirofilaria), Ehrlichia spp, Anaplasma spp, Borrelia burgdorferi
- **trūRapid Giardia**  
Giardia duodenalis
- **trūRapid Parvovirus**  
Canine Parvovirus

## SIMPLE 3 STEP PROCESS

